




# ZR<sup>®</sup> Saline Flush Syringes

## HEPARIN LOCK SYRINGES

The ZR<sup>®</sup> Difference




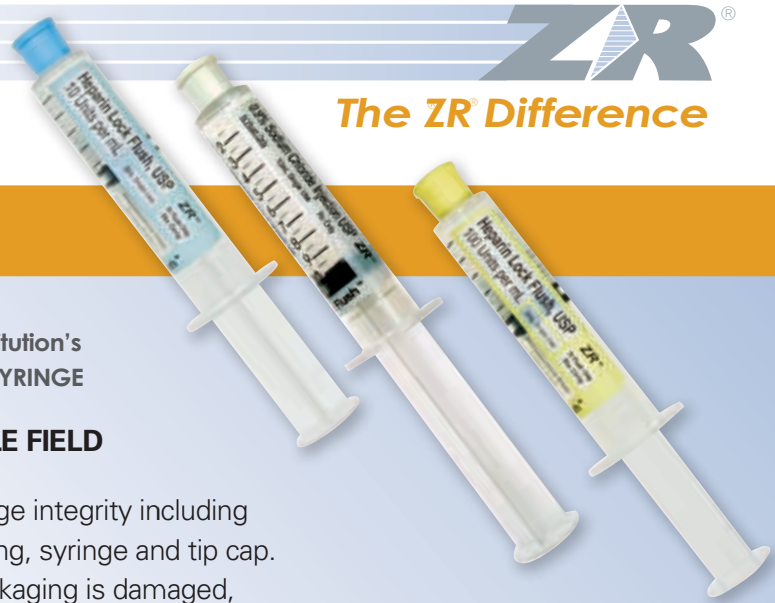


# Flush Procedure Guide

Use aseptically – For IV flush only.

Follow Excelsior Medical's instructions for use and your institution's procedures for IV flush administration. **DO NOT REUSE ANY SYRINGE**

### NOT FOR STERILE FIELD

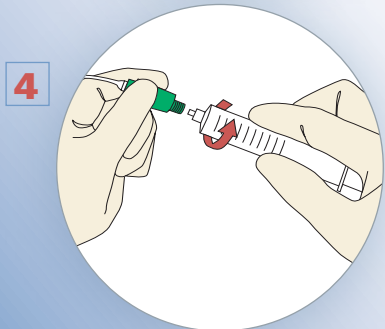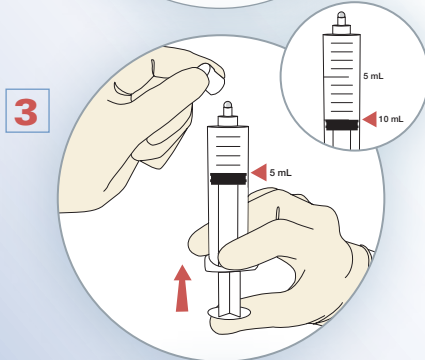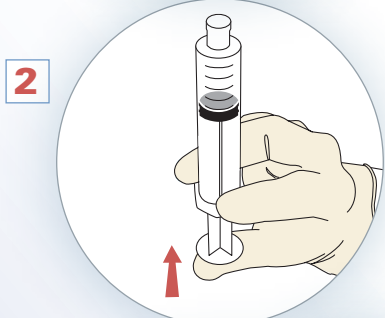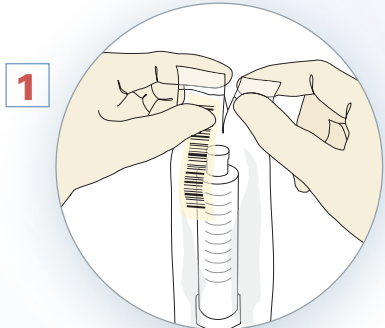

- 1** Inspect package integrity including protective wrapping, syringe and tip cap. Do Not Use if packaging is damaged, syringe is leaking or tip cap is not fully attached to syringe. Remove protective wrapping. Visually inspect the syringe and solution. Do not use if cloudy or hazy. Do Not Use if the syringe/tip cap is damaged, syringe is leaking or tip cap is not fully attached to syringe.
- 2** **Before removing cap**, hold syringe with tip cap upright. Press syringe plunger until the plunger moves slightly. This motion breaks the seal between the syringe plunger and cylinder and allows smooth administration of solution.
- 3** Remove tip cap with a twisting motion from the end of the syringe. Take care not to contaminate the syringe tip. Attach blunt cannula or needle if required. Hold syringe with tip upward, tap side for bubbles to rise, apply and maintain pressure to plunger until plunger reaches the labeled flush volume on syringe (e.g., 10mL, 5mL, 3mL etc.) and air is expelled.
- 4** Flush site with an appropriate volume of solution (determined by your institution) for maintaining patency.

Use in accordance with any warnings or precautions appropriate to the medication being administered. Do not use if compatibility is in doubt.

For Single Use Only - Discard syringe and any unused portion of the solution.



Excelsior Medical, LLC  
Corporate Headquarters  
1933 Heck Avenue, Neptune, NJ 07753  
[www.excelsiormedical.com](http://www.excelsiormedical.com)

Toll Free 800-487-4276  
Main Phone 732-776-7525  
Fax 732-776-7600  
E-mail: [info@excelsiormedical.com](mailto:info@excelsiormedical.com)  
ZR<sup>®</sup> is a registered trademark of Excelsior Medical, LLC a Medline Company.